



***Kem Medical Products Corp.***

# **KemSafe™ Solution Neutralizer for Glutaraldehyde and OPA disinfectant solutions**

*(Glycine based formulation)*

**Catalog # 9074**

## **Directions:**

1. Pour **KemSafe™** Solution Neutralizer into container with waste disinfectant solution, which you intend to neutralize (one 32 oz. bottle neutralizer per 2 gallons waste solution or 8 parts waste solution).
2. Reaction is complete in 5 minutes (a color change will occur), the neutralized solution is then safe for disposal.

*Note:* If the **KemSafe™** Solution Neutralizer is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the neutralized disinfectant solution.

Always maintain adequate ventilation in accordance with applicable regulations.

Disposal is subject to federal, state, and municipal sewer regulations.

## **Precaution:**

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.

*Rev 2*

*February 1, 2011*