



# ***Kem Medical Products Corp.***

## **KemSafe™ Powder Neutralizer for Glutaraldehyde and OPA disinfectant solutions**

*(Glycine based formulation)*

**Catalog # 9075** (24 containers/case)

**Catalog # 9075-12** (12 containers/case)

### **Directions:**

1. Pour **KemSafe™** Neutralizing Powder into container with waste disinfectant solution that you intend to neutralize (one container neutralizing powder per 1 gallon waste solution).
2. Stir solution to mix.
3. Reaction is complete in 5 minutes (a color change will occur); the neutralized solution is then safe for disposal...

**Note:** If the KemSafe™ Neutralizer Powder is to be used in a scope processor, the processor must be purged after neutralization has occurred. It is recommended that a water rinse follow the evacuation of the waste solution.

**Note:** Always maintain adequate ventilation in accordance with applicable regulations. Disposal is subject to federal, state, and municipal sewer regulations.

### **Precaution:**

This product is not to be ingested or brought into contact with the skin and eyes. Review manufacturer's Safety Data Sheet. Keep away from children.