



Kem Medical Products Corp.

KemSafe™
Glutaraldehyde and OPA Neutralizing Mats
(Glycine based formulation)

Catalog # 9076/9077

Directions:

Place **KemSafe™** Neutralizing Mat directly on small spills or under soaking containers containing Glutaraldehyde or OPA disinfectant solution. When used as directed, KemSafe™ Neutralizing Mats will absorb and neutralize accidental small spills and splashes from soaking bins containing the disinfectant solution in 5 minutes.

Note: KemSafe™ Neutralizing Mats are effective only upon the initial absorption of the disinfectant solution. Areas already saturated will not be neutralized.

Replace KemSafe™ Neutralizing Mats daily or after the majority of the area of a given mat has come in contact with disinfectant solution.

Precaution:

Gloves are recommended. Prolonged skin contact can cause irritation. Review manufacturer's Safety Data Sheet. Keep away from children.